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Product data sheet Face mask „AllegraMask® PRO“ PPE

1. Product name

Face protection mask (mouth and nose protection) „AllegraMask® PRO“ PPE; Personal protective equipment according to EU Regulation 2016/425, Annex I Category I and Annex IV

2. Intended use / areas of application

The face protection mask „AllegraMask® PRO“ protects against drops and splashes of liquids. In normal breathing, it creates a dense barrier between the users and can therefore also be used as additional protection, for example in the medical area in addition to the surgical mask. The face mask „AllegraMask® PRO“ protects against weakly aggressive cleaning agents in the face area (mouth and nose area) that are used in the professional or private environment. The „AllegraMask® PRO“ face mask also protects against superficial mechanical injuries. The area of application of the „AllegraMask® PRO“ is in the medical field, nursing, rescue, catering, in workshops and all affected areas of work that share the same above Need protection purpose. It can also be used for a variety of areas in public life, such as B. in school, in supervision, in advice, in service or in sales.

3. Description of function

The face protection mask „AllegraMask® PRO“ is placed over the nose-and-mouth area like glasses. The fit should completely cover the mouth and nose area. If this is not the case, the mask should not be used and the manufacturer should be contacted.

4. Safety instructions / contraindications

For your own safety and the safety of third parties, please note the following points:

- ATTENTION:** The face mask „AllegraMask® PRO“ does not offer any protection against biological agents.
- ATTENTION:** Before first use, please clean the AllegraMask and check before putting it on whether all parts are stuck and undamaged.
- ATTENTION:** The „AllegraMask® PRO“ only protects in the covered area. Possibly eye protection must also be used.
- ATTENTION:** Check the face mask for damage and correct fit before each use.
- Only original spare parts may be used, otherwise the function or biocompatibility may be restricted.
- ATTENTION:** Not suitable for small children, e. g. because of small parts that can be swallowed.
- To avoid infection or bacterial contamination, please refer to the section “Cleaning and disinfection”
- ATTENTION:** The face mask is delivered with a tether. Please note the UVV regulations that apply to your use. When working on machines etc., there is a risk of injury if there is a strong pull. Please attach the tether so that it detaches from the bracket when you pull it gently.
- The „AllegraMask® PRO“ should not be used for safety-relevant activities with a detailed view downwards, as the field of vision may be impaired.
- Harmlessness: The materials used are not known to be harmful to the user, cause skin irritation or breathing difficulties. However, materials that come into contact with the wearer’s skin may cause allergic reactions in susceptible individuals. In these cases, do not use the mask and contact the manufacturer.
- If you feel uncomfortable wearing the mask for a longer period of time, remove the mask, stop using it and contact the manufacturer.
- The face protection mask “AllegraMask® PRO” offers no protection against mechanical impact, shocks, splinters, mechanical parts and the like.
- The “AllegraMask® PRO” face protection mask should not be used or replaced if it is cracked or otherwise damaged. Shows changes in shape and color, has been exposed to chemicals, oils or fats, has been exposed to extreme heat or long exposure to the sun. Damage may not be visible to the end user.

5. Cleaning and disinfection

Please clean or disinfect the mask after each use. The face protection mask should be disinfected when changing the user.

5.1. Cleaning

Wash all parts in warm water using a pH-neutral solution, soap solution or with an alkaline cleaning agent; then rinse under warm running water to remove residues. Strongly alkaline, acidic or alcoholic media as well as hard brushes, dry cleaning with paper, abrasive cleaners or other materials that can damage the surfaces must not be used.

5.2. disinfection

A disinfection of all plastic parts made of MABS, SEBS or polycarbonate can be carried out with commercially available disinfectants suitable for plastic materials. Alcohol-based disinfectants should not be used, as otherwise the coating may be damaged. The manufacturer’s information on exposure times or Application concentrations of disinfectants must be strictly adhered to. Please also observe the safety instructions of the disinfectant of the manufacturer.

6. Anti-fog and anti-static agents

Anti-fogging and anti-static agents can be used as useful supplements. When using, only the means specified by us should be used. In this case, also observe the manufacturer’s safety instructions.

7. Technical data / storage / transport

Storage temperature: - 40 ° C to 60 ° C Note: liquid media may only be stored above 0 degrees Celsius

Operating temperature: -18 ° C to 50 ° C

Duration of use: It is recommended to wear the face mask “AllegraMask® PRO” for a maximum of two hours without interruption.

Storage: Storage is recommended in laundry or cotton bags. Don’t leave it in the sun.

8. Materials

1	Bracket	MABS
2	Nose pad	SEBS
3	Visor	PC
4	Strap	Cotton

9. Spare parts / accessories

- Bracket
- protective shield
- strap
- nose pad

10. Disclaimer of liability

To the best of our knowledge, the information above is correct. However, it is not claimed that these are complete and it should therefore only be considered as a guideline. The information provided here is intended to describe the product with regard to the necessary safety precautions. It is not intended to guarantee certain properties, is based on the current state of our knowledge and does not constitute a contractual legal relationship. The company cannot be held responsible for any damage caused by handling or contact with the above product. Liability excluded.

11. Legenda

	Latex free		Protect from heat and light
	Not for reuse		Order number
	Follow the instructions		Batch name
	Best before [date]		Contains phthalate
	Manufactured on		Phthalate free

FAIST MEKATRONIC SRL
Nicolae Filipescu 18 | RO - 410605 Oradea
E-Mail: info@allegramask.com| www.allegramask.com
Phone +40 359 803 600 | Fax +40 359 803 602

Conformity according to PPE EU regulation 2016/425

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